

Installation Instructions

Ergotron Backframe Mount Monitor Arm


Tools Needed:

- 1/2" socket
- #2 phillips head screwdriver

Parts that are included in kit #805-0003:



Inner and outer backframe mounting brackets with mounting bolts

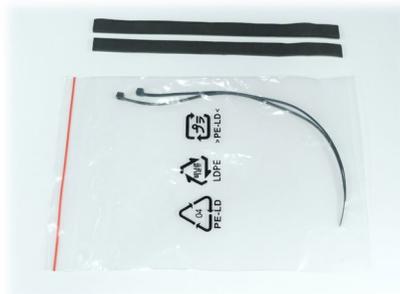

Foam tape and zip ties



Monitor bracket with monitor bracket covers



Monitor arm extension



Monitor arm with cap



Allen wrenches and screws packet


Installation Instructions

Ergotron Backframe Mount Monitor Arm

Installation Instructions:



Step 1- Install foam tape on the edges of the outer backframe mounting bracket.

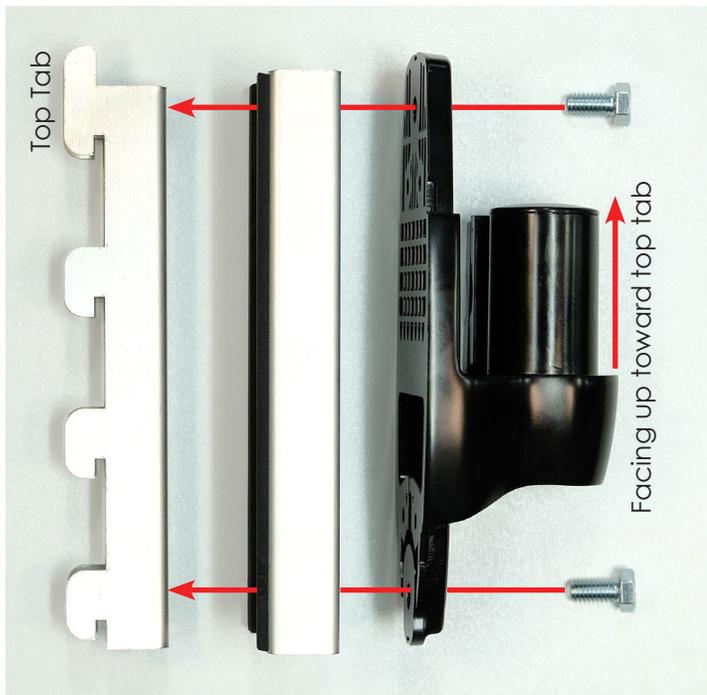

Step 2- With the top tab facing up, insert bolts through monitor bracket, then outer backframe mounting bracket then into the welded nut on the inner backframe mounting bracket. Hand tighten until bolt is about half way in (keep it loose).




Step 3- Insert top tab upwards into backframe slot.

Installation Instructions

Ergotron Backframe Mount Monitor Arm




Step 4- Push forward onto the backframe.

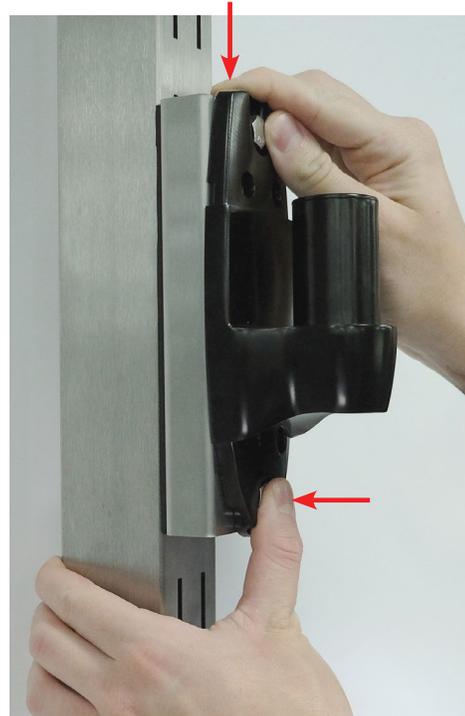

Step 5- Press lower bolt while pushing down on the monitor mount until tabs are set into backframe slots.



Step 6- Verify that the tabs are completely seated in slots.

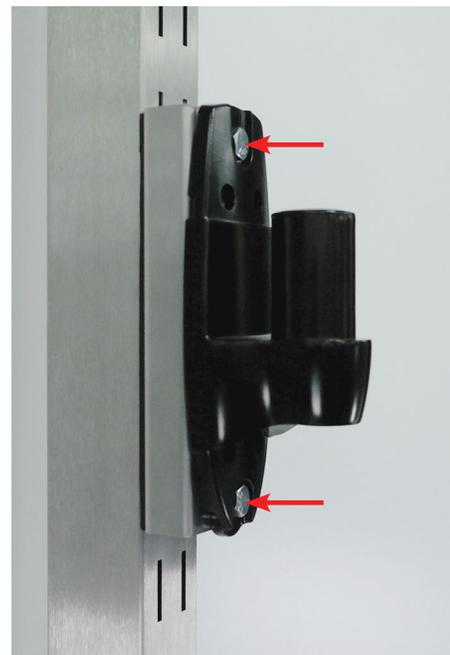

Step 7- Tighten bolts with 1/2" socket.

Installation Instructions


Ergotron Backframe Mount Monitor Arm



Step 8a- Set monitor arm extension on monitor bracket if you want a 2 arm monitor mount, then move to step **Step 9**. If you want a single arm setup, see **Step 8b** below.



Step 9- Set the monitor arm on monitor arm extension and insert the cap. Then, slide the monitor mount covers over bolts.

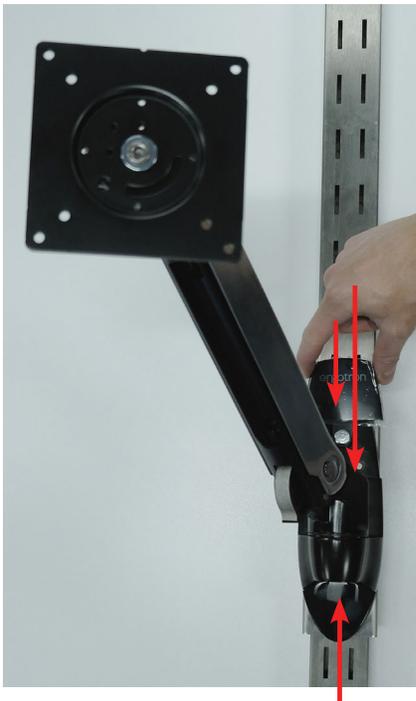

Step 8b- For a single arm monitor mount, set monitor arm on monitor bracket and insert the cap. Then, slide the monitor mount covers over bolts.

See the included Ergotron LX Wall Mount Monitor Arm User Guide for instructions on how to mount your monitor, run the cables, and adjust the arms.