

# Installation Instructions

## Drawer for Prep and Pack Workstations

### Installing a Drawer Assembly to a Prep and Pack Workstation Without a Drawer

#### Tools Needed:

- 7/16" socket

#### Top Drawer Assembly Kit:

- 1 Drawer/ 15" W x 14" D x 3.5" H
- 1 Drawer Box
- 4 Bolts
- 4 Mounting Brackets
- 2 Keys



**Step 1-** Remove the drawer from the drawer box by pressing the lever up or down (arrow stamped on slide - see red circle) while pulling drawer out.




# Installation Instructions

## Drawer for Prep and Pack Workstations

**Step 2-** Insert bracket with nut into slots (shown with red circles) under the prep and pack workstation in the direction shown.



View from underneath

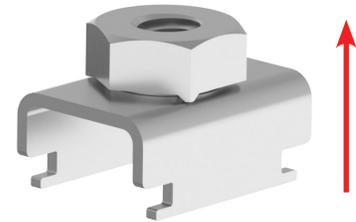

Bracket with Nut

**Step 3-** Slide the brackets to the position needed to mount the drawer box (shown with red circles).



View from underneath

# Installation Instructions

## Drawer for Prep and Pack Workstations

**Step 4-** Line up and install bolts through the nut welded to the bracket. Hand tighten to keep in place. Using a 7/16" socket, tighten bolts.



**Step 5-** Line up slides on drawer with slides on drawer box and push drawer back in.



**You have finished the installation of your top drawer.**

# Installation Instructions

## Drawer for Prep and Pack Workstations

### Installing an Additional Drawer Assembly to a Prep and Pack Workstation with an Existing Drawer (Recommended 2 Person Job)

#### Tools Needed:

- 7/16" socket and a 7/16" open end wrench

#### Additional Drawer Assembly Kit:

- 1 Drawer/ each 15" W x 14" D x 3.5" H
- 1 Drawer Box
- 4 Bolts
- 4 Lock Nuts
- 2 Keys




**Step 1-** Remove the drawers from each drawer box by pressing the lever up or down (arrow stamped on slide - see red circle) while pulling drawer out.



# Installation Instructions

## Drawer for Prep and Pack Workstations

**Step 2-** Slide lock nuts under drawer slide then center over bottom holes

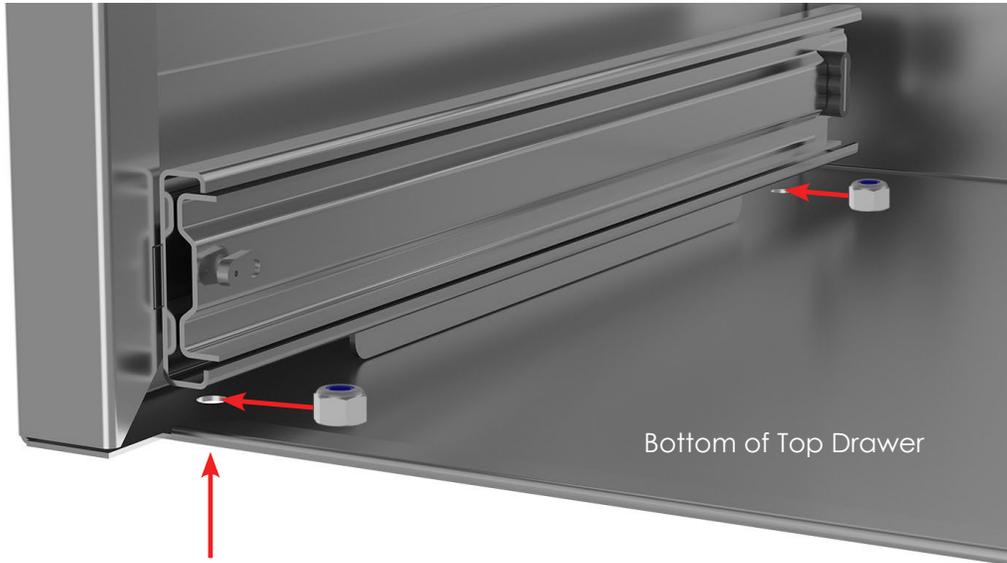


**Step 3-** Have another person lift the drawer box and align the holes at the top of the additional drawer with the holes at the bottom of the drawer above it. Insert bolt upwards through hole into lock nut then hand tighten. Now do the same to the other side.



# Installation Instructions

## Drawer for Prep and Pack Workstations

**Step 4-** Slide open end 7/16" wrench over lock nut then tighten with a 7/16" socket from beneath. Do this for all 4 bolts.

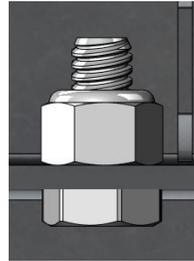

**Step 5-** Line up slides on drawer with slides on drawer box and push drawer back in (for both drawers).



You have finished the installation of an additional drawer.

Repeat this process for additional drawers.

It is not recommended to install more than 4 drawers on a single table.

It is not recommended to install more than 1 drawer on the same side as a CPU holder as it would limit the size of the CPU.